# Service Pack CEREC Premium SW 4.4.3

## Update Description (EN)

### Installation notes:

- Perform a backup of all patient data before starting the installation process.
- Do not perform an installation on several CEREC units or computers at the same time if these are within the same network.
- Before starting the installation process, please check the patient database directory in the software (Configuration/Settings/Patient Database). The path should always point to a directory, never to the root of a drive, i. e. never to "D:\", but rather to "D:\Data".
- If you are running a CEREC Premium SW 4.4.x and an inLab SW 15.1 software on your PC, please make sure that both products are installed in different directories. This can be achieved by performing custom installations of the products.

### Changes:

- Sirona ScanPost can be selected for Straumann Variobase®, (with the exception of NNC Variobase® C 3.5)
- CEREC MC XL: Fix for "unknown communication error"
- CEREC SpeedFire:
  - Improvements to Wi-Fi configuration
  - App stability ("black screen" issue) improved
- Shape Tool: Improved results near preparation margin causes
  - Custom parameter presets
  - Favorite materials

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