# CEREC Ortho SW 2.0

## Update Description (EN)

### Installation notes:


- o Perform a backup of all patient data before starting the installation process.
- o Do not perform an installation on several acquisition units or computers at the same time if these are within the same network.
- o Before starting the installation process, please check the patient database directory in the software (Configuration/Settings/Patient Database). The path should always point to a directory, never to the root of a drive, i. e. never to "D:\", but rather to "D:\Data".

### Hardware Support:

CEREC Ortho SW 2.0 supports Omnicam (with guided scanning) and Primescan (no guided scan required) scanners, but requires the newer AC models with touchscreen and Windows 10 to run. An upgrade version for existing Omnicam installations will follow at a later date

### Changes:

- o New user interface design to coincide with the new generation of CAD/CAM software (i.e. CEREC and Connect SW 5)

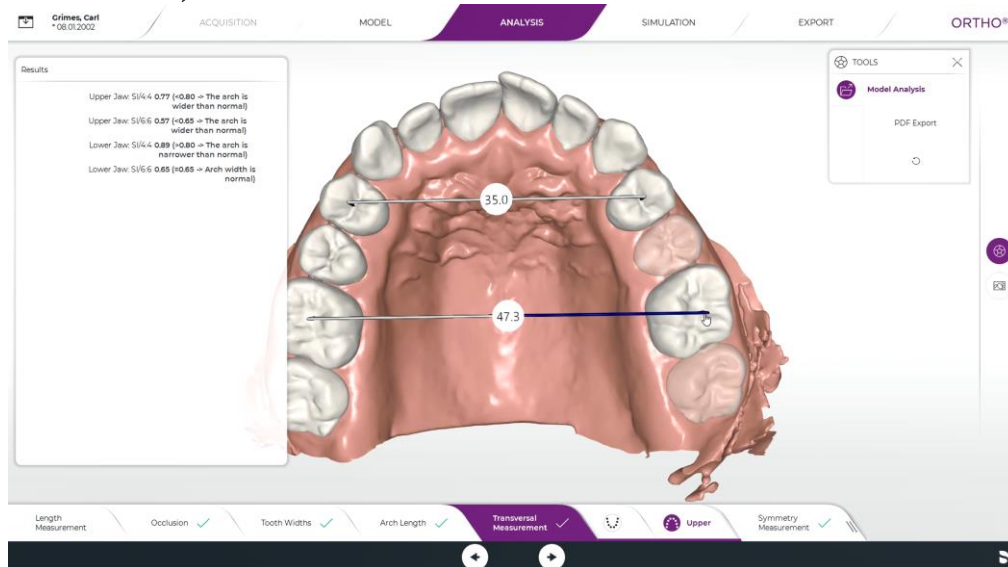- o Support for Primescan and newer AC units  
This version has been improved to support interaction via the touchscreen of the current AC, and expanded to allow scanning with Primescan, without the need for guided scanning.
- o Improved scanning performance for Omnicam via a complete rebuild of the guided scanning process
- o Treatment Simulation: The software includes a new phase called SIMULATION, with the main goal of simulating the effects of orthodontic treatment during chairside use of the software, thus aiding patient consultation.  
Once the teeth have been marked and segmented in the MODEL phase, SIMULATION becomes available. Upon entering this new phase, the treatment simulation is automatically shown. Use the "Original Position" option in the Tool Menu on the right side of the screen to toggle between the current situation and the treatment simulation.
- o Improved Model Analysis, expanded significantly, now including all common analysis types.  
A new ANALYSIS phase has been added, which is subdivided into multiple steps in which the user sets measuring points on the model. Based on these measuring points, a PDF export can be created which includes the most common model analysis types. The more measuring points set, the more content the PDF will have.
- o Seamless export to SureSmile: Patient cases can be uploaded directly into the user's SureSmile account.

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